1

Angelegen Positioning

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INFORMATION NOTICE AND USER MANUAL

The **CE** Marking on your **ONB ORTEMA Neck Brace** means that it has been submitted to the EU-Type Examination process in accordance with the requirements of **Regulation (EU) 2016/425** and has been granted a 2nd CAT. PPE (Personal Protective Equipment) against medium risks.

The performances of your **ONB** have been verified by **RICOTEST** Notified Body nr. 0498 (Via Tione 9, 37010 Pastrengo- Verona - Italy) on the basis of a laboratory testing procedure performed under the framework of the "Internal Method 088".

The EU Declaration of Conformity can be accessed at the following link:

www.ortema-shop.com/euconform

Intended Usage

The **ONB** is a preventive measure that is intended to stabilize the athlete's neck during sporting activities and to prevent injuries in the case of an accident.

The **ONB** developed in Markgröningen is an effective means of closing the safety gap between helmet and back protector considering biomechanical criteria. It can be used in Motocross, Enduro, Supermoto, Kart racing and generally speaking riding

on two wheels. Core piece of this extremely light-weight construction is the ORTEMA typical horseshoe shaped support in the back. In the case of an impact the forces can be evenly distributed over the shoulder blades and be resorbed by the force resorbing frame. This feature prevents a direct contact of the neck protection system with the spine and a pressure build-up on the vertebrae in the case of overextending.

The **ONB** was mutually developed with spine surgeons and orthopaedic surgeons. It prevents the hyperextension of the neck and by material deformation the extreme movements of the head are progressively reduced which can prevent injuries to the cervical and thoracic region of the spine. The extremely low weight, no contact to the collarbones and the hinge in the front of the brace result in great wearing comfort which fully meets the needs of athletes on two wheels.

The **ONB** is available in 5 sizes and the height can be easily and quickly adjusted with the shoulder pads. Furthermore you can choose from standard decals in different colours.

To correctly use the **ONB** we advise you to thoroughly read the following instructions and to adhere to these when using the brace. To further optimize the protection of the **ONB** we recommend you also use our back and chest protector.

Practical tips

- Open the straps on the side and in the front.
- Slip one arm through the side strap and put the ONB around your neck with the front opened. Then slip the other arm through the second belt on the side.
- First close the main ring clip and then the clip on the front strap.
- Take care that the ONB rests properly on the shoulders and is not twisted. Using the straps on the side the ONB is adjusted for a proper and comfortable fit.
- The ONB should have proper contact to the region of the chest, shoulder and back.
- By moving your head with the helmet on check for proper fit and also check the height between shoulder and helmet with the variable pads.
- After correctly adjusting the system sufficient freedom of movement should still be possible.

Adjusting the shoulder support height

- To adjust the side support height the **ONB** is delivered with pads that differ in strength. The needed pads can be fixed to the brace quickly and efficiently by the Velcro fastening. After putting on

the **ONB** and testing the freedom of movement the brace should have sufficient contact for stabilization. Should the proper adjustment of the height not be possible with the delivered pads please contact ORTEMA for advice.

- Fit all adjustable straps in such a way so as to avoid any slipping while using.

Control

Check the **ONB** for correct positioning, if necessary also visual control by a second person.

Correct fit assessment

The neck brace is designed for normal body measurements. Should the athlete differ strongly from the norm additional fitting measures might be necessary. Should you encounter problems with the fit we would ask you to contact your authorized retailer or ORTEMA directly to find a solution.

Choosing the correct size

The correct choice is mainly determined by body height and body weight. Look at the table to determine your size.

Care and maintenance

- After use the ONB can be cleaned by hand washing with cold water and a mild non-detergent soap.
- Let it dry without exposing unnecessarily to the sun or any heat sources.
- Normal use requires the **ONB** to be inspected after a period of 12 months at the latest. This service can only be performed by ORTEMA or a certified retailer. Should this inspection not be performed all possible warranty claims regarding wear and tear are no longer applicable.

Warranty

The warranty period is 2 years and applies to the **ONB** corpus (not the strap system or parts subject to regular wear and tear).

No warranty claims are possible should the following be applicable:

- Damage caused by misuse of the **ONB**
- Damage caused by falling, collisions or any other accidents where the corpus is forcefully stressed. In this case the **ONB** should be exchanged immediately anyway

Exclusion criteria

- The **ONB** is not intended for use for clients that already have bone injuries or spinal injuries and is not to be used after spinal cord surgery.
- The Neck Brace System is no guarantee that injuries can be completely prevented when participating in sports.
- Fittings of the brace to meet your personal anatomical requirements, differing from the standard, may only be performed by ORTEMA itself or a ORTEMA certified company.
- Products modifications result in immediate loss of all warranty claims
- Changes and alterations to the corpus, strap system or any other features may only be performed by the manufacturer

Notes on storage

- Prevent the **ONB** of being exposed to direct sunlight and store it in a dry place.
- Never leave the **ONB** in the car with high outside temperatures and prevent the ONB from being exposed to high temperatures generally.

Additional information / Disclaimer

The **ONB** can rely upon years of experience gathered by ORTEMA in development and production. It was developed with material technology specialists and engineers specialized in production techniques and is used primarily as a preventive measure.

The information in this instruction manual is important for your own safety. Please keep this information for later use. The **ONB** is not a medical device and not intended for the treatment of injuries of the cervical spinal column. It is solely intended as a preventive measure to reduce the risk of neck injuries. Even though wearing the **ONB** the risk of severe injuries cannot be eliminated when falling, experiencing a hard blow or similar accidents where high forces are involved.

The **ONB** is intended to protect the athlete while participating in higher-risk sporting activities and is intended to help prevent injuries. Should wearing or putting on the **ONB** brace create any problems whatsoever immediately remove it.

Should you experience any problems regarding the proper fit please contact ORTEMA or a ORTEMA certified dealer. Here additional information can be obtained to solve any possible problems regarding the proper use.

Note

Not taking care of your brace can also result in skin irritations and/or abrasion of the skin. Taking proper care can also prolong the lifespan of the product significantly. The lifespan of the brace is however limited. After an accident or blow or obvious wear and tear of the corpus and/or strap system the brace should be replaced and not used any longer. To correctly assess the condition of the brace let it be checked either by ORTEMA or a ORTEMA certified dealer.

Limited warranty

The **ONB** has a limited warranty regarding material and workmanship. No warranty claims can be submitted for wear and tear parts such as belts, padding or any movable parts.

No guarantee exists for damages caused by accidents, misuse, carelessness or regular wear caused by use of the brace. Abrasions, matted surfaces, scratches and edges resulting from use are not entitled to a guarantee. Should you have any doubts as to the proper functionality of the brace you can send it to ORTEMA for inspection.

Please note that the brace should be cleaned before sending. Should the brace have to be cleaned by ORTEMA this results in additional costs. Repairs may only be performed using original parts supplied by ORTEMA. Shipping thereof is paid by the person ordering.

ORTEMA cannot be made accountable for the loss of any packages sent to us containing an ONB which is to be checked, repaired or is sent back in the